

**Corsinel®**  
by Tytex

SUPPORT BELT



HIGHLY BREATHABLE



4-WAY STRETCH



SEAMLESS KNITTING



FULLY OPENABLE



LATEX FREE

# OSTOMY AND HERNIA SUPPORT BELT



## ABOUT SUPPORT BELT

- ✕ OFFERS DISCRETION AND SECURE FIT
- ✕ SOFT, SEAMLESS AND BREATHABLE FOR OPTIMUM COMFORT
- ✕ **MAXIMUM SUPPORT** FOR OPTIMAL HERNIA MANAGEMENT
- ✕ POCKET FOR EASY APPLICATION



STANDARD  
100

6234-7001  
DTI

EUROPEAN-BASED PRODUCTION

FOR OPTIMAL QUALITY,  
SUPPLY CHAIN EFFICIENCY  
AND FLEXIBILITY



# Corsinel®

by Tytex

## CORSINEL BELT

ARTICLE NUMBERS: 3518, 3519 AND 3520




### AVAILABLE COLOURS



### COMPOSITION

54 % PA, 33 % CO,  
13 % EL


### CARE HANDLING

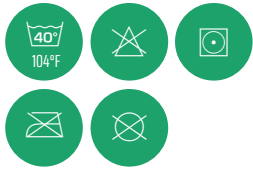

- Close hook-and-loop before washing
- Wash with similar colours and in a washing bag
- Use of chlorine based bleach or chlorine based disinfection will decrease durability and function



### AVAILABLE IN SIZES

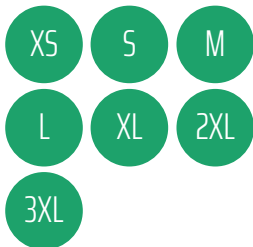

### PRODUCT RANGE

Belt 15 cm, art. no. 3518  
Belt 20 cm, art. no. 3519  
Belt 26 cm, art. no. 3520

### NOTE

For more support go down one size and for less support go up one size.

IMPORTANT: Always measure yourself to find the right size.

### ABOUT THE PRODUCTS

The Corsinel Belts are designed to fixate and support ostomy pouches and/or hernias. The belts offers maximum support, making daily and strenuous activities easier to perform. Use of the belt also enhances a smooth appearance, providing discretion and comfort.

The belts have 4-way-stretch, which provides perfect adaption to body contour. Seamless knitting offers optimum wearer comfort. The belts are designed for all day use.

### FEATURES

- 4-way stretch: freedom of movement and supportive fit
- Breathable materials designed for air permeable and moisture absorbing qualities
- The device is seamlessly knitted in one piece
- Breathable materials designed for air permeable and moisture absorbing qualities
- Silicone anti-slip pattern in the waistband securing that the device stays in place
- Ostomy pouch safely fixated during physical activity
- Tested for harmful substances according to OEKO-TEX STANDARD 100 class II
- Easy-to-apply pocket and wide hook-and-loop for adjustment of support level
- Low cut back

### INDICATION

For postsurgical use after construction of a colostomy, an ileostomy or an urostomy. For management of parastomal- or other abdominal hernias. For use with both 1- and 2-piece systems.

### PRECAUTION

- Follow the Instructions for Use
- Follow recommendations given by your doctor or ostomy nurse
- Use the device according to the indications to avoid injury or damage
- Make sure the device is not compromising stoma and/or lung functions
- If your hernia is irreducible (cannot be returned by manipulation), seek the advice and guidance of your ostomy nurse before using the device
- If any unexpected discomfort, stoma or skin irritation occurs, remove the device
- By urgent need for removal of the device, open with scissors
- Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size if you prefer extra tight-fitting garments, and the larger size if you prefer the garment less tight
- Ensure that the garment feels comfortable to wear when sitting and standing
- The device may contain traces of softening and antistatic agents

### RECOMMENDED SIZES

15 - 20 CM

26 CM

SIZE	CM	IN	SIZE	CM	IN
XS	74 - 82	29 - 32	M	86 - 98	34 - 39
S	78 - 90	31 - 35	L	94 - 106	37 - 42
M	86 - 98	34 - 39	XL	102 - 116	40 - 46
L	94 - 106	37 - 42	2XL	112 - 130	44 - 51
XL	102 - 116	40 - 46	3XL	126 - 148	50 - 58
2XL	112 - 130	44 - 51	-	-	-
3XL	126 - 148	50 - 58	-	-	-



Measurement at the widest abdominal circumference

WWW.TYTEX.COM

TYTEX A/S HAS BEEN CERTIFIED ACCORDING TO ISO 9001, ISO 14001 AND ISO 13485