

Corsinel®
by Tytex



HIGHLY
BREATHABLE



4-WAY
STRETCH



SEAMLESS
KNITTING



LATEX
FREE

OSTOMY AND HERNIA SUPPORT TUBE

OSTOMY SUPPORT TUBE



ABOUT **CORSINEL OSTOMY SUPPORT TUBE**

- ✕ **MAXIMUM SUPPORT** FOR HERNIA MANAGEMENT
- ✕ OFFERS DISCRETION AND SECURE FIT
- ✕ **SOFT, SEAMLESS AND BREATHABLE** FOR OPTIMUM COMFORT

OEKO
TEX®

**STANDARD
100**

6234-7001
DTI

EUROPEAN-BASED PRODUCTION
FOR OPTIMAL QUALITY,
SUPPLY CHAIN EFFICIENCY
AND FLEXIBILITY



Corsinel®

by Tytex



TUBE

ARTICLE NUMBER: 3515, 3516, 3517


AVAILABLE COLOURS



COMPOSITION

80% PA, 20% EL

CARE HANDLING

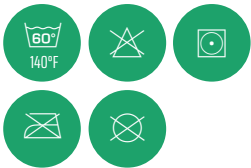

- Wash with similar colours and in a washing bag
- Use of chlorine based bleach or chlorine based disinfection will decrease durability and function



AVAILABLE IN SIZES



PRODUCT RANGE

Tube 15 cm, art. no. 3515
 Tube 20 cm, art. no. 3516
 Tube 26 cm, art. no. 3517

NOTE

For more support go down one size and for less support go up one size.

IMPORTANT: Always measure yourself to find the right size.

ABOUT THE PRODUCTS

Corsinel Tube is designed to support ostomy pouches and/or hernias. The tube offers maximum support, making daily and strenuous activities easier to perform. Use of the tube also enhances a smooth appearance, providing discretion and comfort.

The tube has 4-way-stretch, which provides perfect adaption to body contour. Seamless knitting offers optimum wearer comfort. The tube is designed for all day use.

FEATURES

- 4-way stretch: freedom of movement and supportive fit
- Breathable materials designed for air permeable and moisture absorbing qualities
- The device is seamlessly knitted in one piece
- Knitted in soft and comfortable material
- Breathable materials designed for air permeable and moisture absorbing qualities
- Silicone anti-slip pattern in the waistband securing that the device stays in place
- Ostomy pouch safely fixated during physical activity
- For 24 hour use if maximum support is tolerated at night
- Tested for harmful substances according to OEKO-TEX STANDARD 100 class II

INDICATION

For postsurgical use after construction of a colostomy, an ileostomy or an urostomy. For management of parastomal- or other abdominal hernias. For use with both 1- and 2-piece systems.

PRECAUTION

- Follow the Instructions for Use
- Follow recommendations given by your doctor or ostomy nurse
- Use the device according to the indications to avoid injury or damage
- Make sure the device is not compromising stoma and/or lung functions
- If your hernia is irreducible (cannot be returned by manipulation), seek the advice and guidance of your ostomy nurse before using the device
- If any unexpected discomfort, stoma or skin irritation occurs, remove the device
- By urgent need for removal of the device, open with scissors
- Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size if you prefer an extra tight-fitting garment, and the larger size if you prefer the underwear less tight
- Ensure that the underwear feels comfortable to wear, when sitting and standing
- The device may contain traces of softening and antistatic agents

RECOMMENDED SIZE 15 - 20 CM RECOMMENDED SIZE 26 CM

SIZE	CM	IN	SIZE	CM	IN
XS	74 - 82	29 - 32	M	86 - 98	34 - 39
S	78 - 90	31 - 35	L	94 - 106	37 - 42
M	86 - 98	34 - 39	XL	102 - 116	40 - 46
L	94 - 106	37 - 42	2XL	112 - 130	44 - 51
XL	102 - 116	40 - 46	3XL	126 - 148	50 - 58
2XL	112 - 130	44 - 51	-	-	-
3XL	126 - 148	50 - 58	-	-	-



Measurement at the widest abdominal circumference

WWW.TYTEX.COM

CE TYTEX A/S HAS BEEN CERTIFIED ACCORDING TO ISO 9001, ISO 14001 AND ISO 13485