

POST-OP RECOVERY BRA FOR  
POSTSURGICAL USE AFTER  
BREAST SURGERY

# MARIANNE



## ABOUT MARIANNE ↘

- › No irritating care label. Care instructions knitted in the bra
- › Suitable when a firm level of compression is recommended
- › Front opening and openable shoulder straps for easy application when shoulder movement may be limited
- › Bilateral pockets for textile breast puffs



4-WAY STRETCH



SEAMLESS KNITTING



TARGETED COMPRESSION  
AND FIXATION



HIGHLY BREATHABLE



FULLY OPENABLE

EUROPEAN-BASED PRODUCTION  
FOR OPTIMAL QUALITY,  
SUPPLY CHAIN EFFICIENCY  
AND FLEXIBILITY



STANDARD  
100  
6234-7001  
DTI



ARTICLE NUMBER: 3373

## MARIANNE

### PRODUCT DESCRIPTION

Marianne bra is designed for the time following a surgical procedure to the breast, when the skin is tender, sensitive and where support is recommended. The bra effectively fixate bandages and wound dressings worn in the post surgical phase. It securely holds drainage bulbs, due to the detachable straps. With a high back and high side panels, it provides comprehensive protection and compression. The Marianne bra support the users breasts during the recovery phase and ensuring optimal user comfort. A seamless design ensures no pressure is exerted on the wound and scar area. The bilateral pockets are perfect for textile breast puffs, allowing the user to create the ideal breast shape and achieve symmetry. With the front opening and openable shoulder straps, it is easy to apply the bra and it allows easy examination of the surgical site.

### PRECAUTIONS

- Follow the Instructions for Use
- Follow your surgeon's recommendations regarding use and fitting
- If any unexpected discomfort or wound irritation occurs, remove the Post-Surgical Bra and contact your surgeon
- The Post-Surgical Bra is not intended for direct contact with injured skin or mucous membrane
- Make sure that bandages, textile breast prosthesis and/or drainage tubes are placed correctly to avoid pressure points and/or obstructed flow
- Before X-ray examination remove Post-Surgical Bra
- Wearing an undersized Post-Surgical Bra may cause local pressure pain and disruption of blood flow and/or lymphatic flow
- Wearing an oversized Post-Surgical Bra may reduce the clinical effect
- Use the Post-Surgical Bra according to medical indication to avoid injury or damage.
- The device may contain traces of softening and antistatic agents



### FEATURES

- Front closure with hook & eye
- Adjustable shoulder straps with A-slide
- High back for rear thoracic support
- High side panels
- Pockets for textile breast puffs
- Tested for harmful substances according to OEKO-TEX STANDARD 100

### INDICATIONS

For use 24 hour per day following; Breast augmentation, breast reduction, breast reconstruction, breast lift (mastopexy), mastectomy, lumpectomy and SN biopsy and axillary dissection of lymph nodes.

### RECOMMENDED SIZES

CM	70	75	80	85	90	95	100	105	110	115
IN	28	30	32	34	36	38	40	42	44	46
AA	S		M			L				
A	S		M			L		XL		2XL
B								XL		2XL
C								XL		2XL
D								XL		2XL
E								XL		2XL

\* **NOTE:** Due to possible postsurgical swelling and edema, you may need to go up 1 - 2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size.  
**Important:** Always measure yourself to find the right size.

### AVAILABLE COLOURS



### COMPOSITION

91% PA  
9% EL

### CARE HANDLING



Close the bra before washing  
Wash with similar colours  
Wash in a laundry bag  
Use of chlorine based products may decrease durability and function

